




The safe, easy and quick way to remove surgical blades.

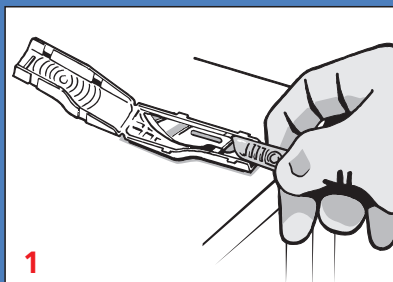


1 Preferably place the open remover onto a stable flat surface and introduce the blade into the bottom section locating it over the blade shape outline.

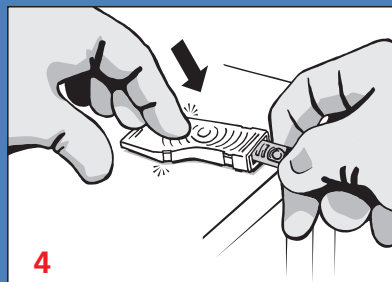


4 Now repeat this with the rear set of locking tabs.

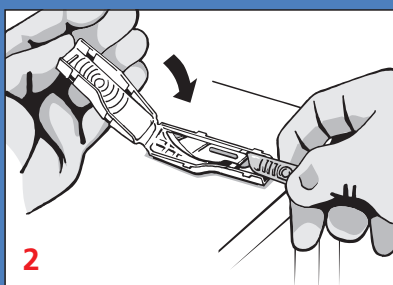


2 With the blade located within the remover use the palm of your other hand to move the top of the unit over and onto the blade until it closes with an audible click.

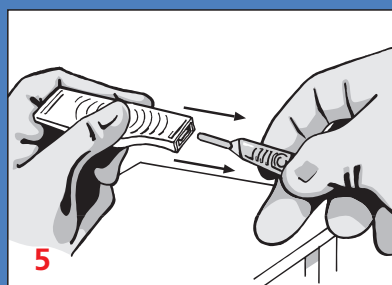

5 Hold the remover with your thumb over the centre of the ring pattern and simply withdraw the handle.

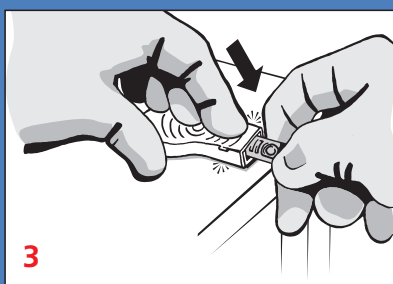

3 Ensure the front set of locking tabs are now engaged by applying downward pressure with your fingers.

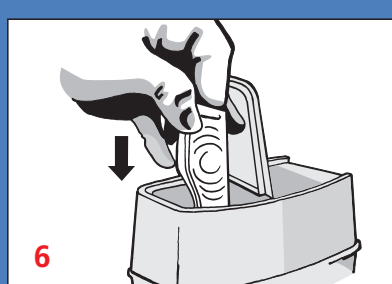

6 The unit can now be disposed of in an appropriate sharps container.

Please refer to the training film available in the Interactive section of our website at <http://www.swann-morton.com/interactive.php>

Swann-Morton and the 'ring pattern' logo are registered trademarks of Swann-Morton Limited and related companies. The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this leaflet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Swann-Morton Limited exclude all liability for injury to the extent permitted by law.

Swann-Morton[®]
Sheffield - England

Swann-Morton Ltd. Penn Works, Owlerton Green, Sheffield S6 2BJ
Telephone: 0114 2344231, Sales 0114 2344223, Fax: 0114 2314966
uksales@swann-morton.com, exportsales@swann-morton.com
orders@swann-morton.com, www.swann-morton.com